



Installation Instructions

reflection series artwall



note


ArtWall installation should conform to local building codes

The installation instructions provided are for the Amico standard products. Slight modifications to the installations may be required. The product shipped to the job site may not be exactly as shown on the installation drawings.


Before Installation

- 1 Review a copy of the final approved shop drawing(s) and/or submittal package. These documents will include technical information specific to the installation, such as:
 - equipment types & quantities
 - room numbers and location
 - shop drawings of each unit type
 - wiring diagrams and rough-in locations
- 2 Locate the headwall unit from the shipping boxes. The contents of each shipping box are listed on the shipping document(s).
- 3 Ensure wall is framed and roughed in as per applicable building standards to accept art wall back box. Allow 1/8" space on each side to allow for adjustment. Back box sizes are as follows:

3-gas box - 30 1/8 w x 21 1/8 h
5-gas box - 42 5/8 w x 21 1/8 h

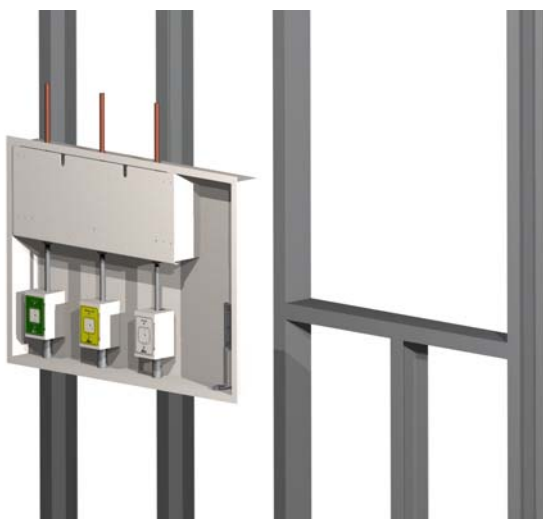


typical example of 3-gas box with 3 gases

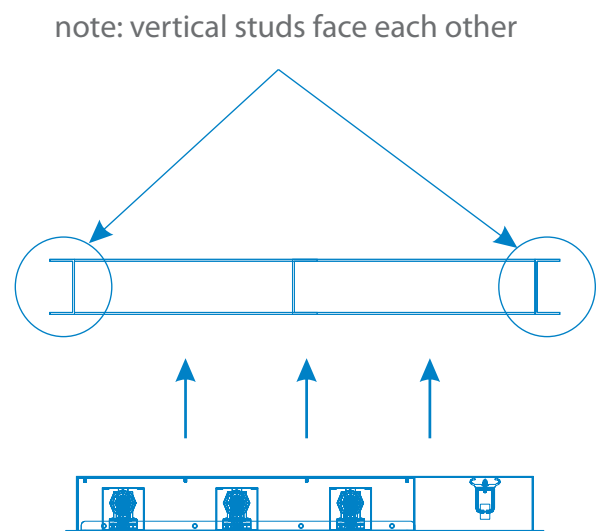


top view

Installation

- 1 Attach art wall back box to wall structure using hardware compliant to applicable building codes. Allow gap between back box flange to stud faces for drywall to be attached later.

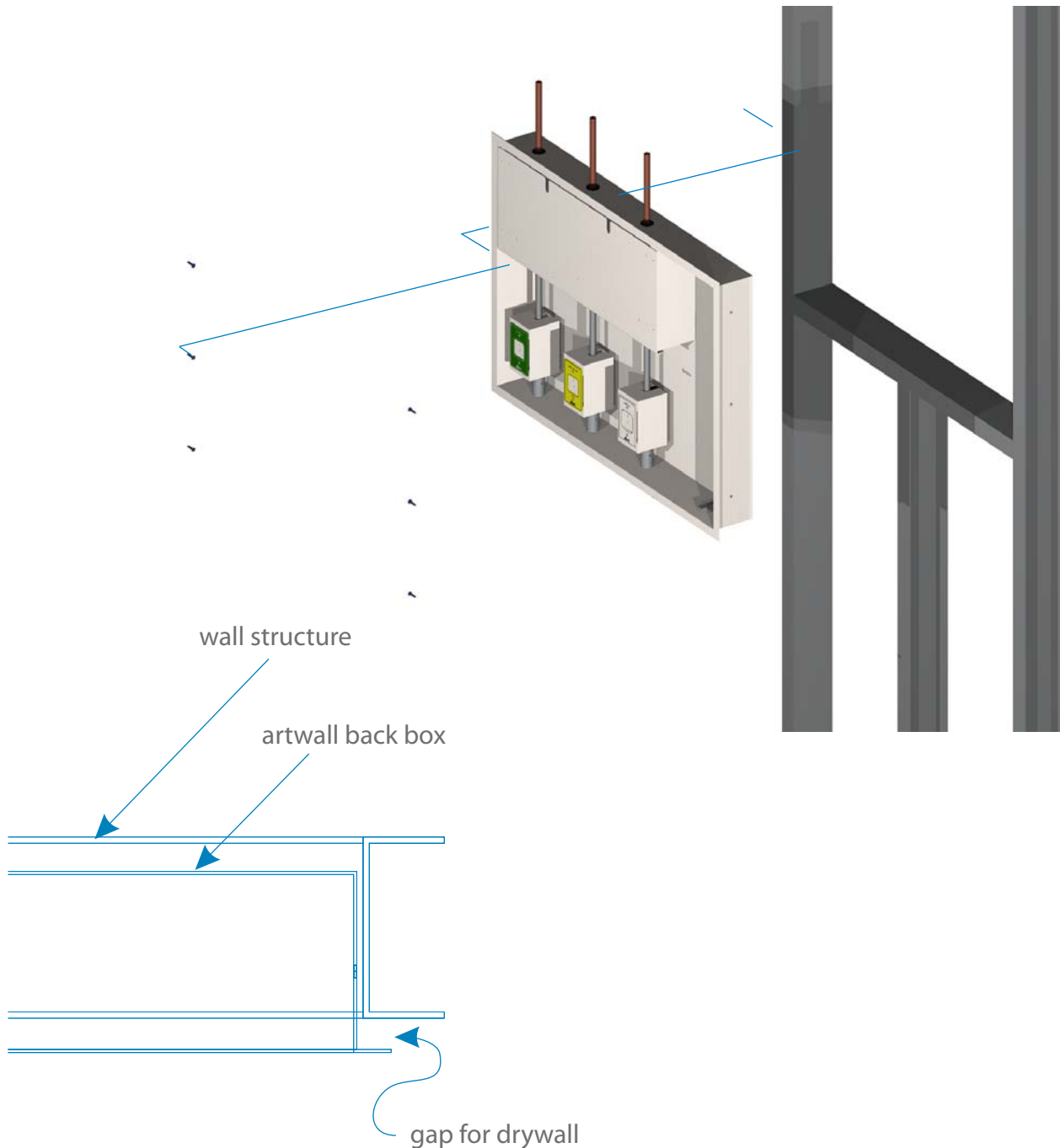

- 2 Make all medical gas connections from art wall to rough-ins. Medical gas pipes are terminated as per shop drawings.

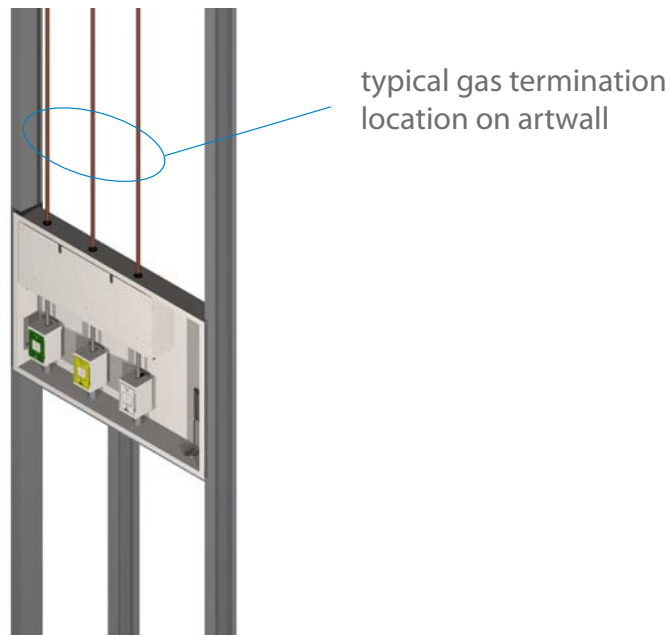


- 3 Insert art into frame, then attach frame to backing plate and movement assembly, using hardware (provided by others). Ensure frame is centered on backing plate. Place assembly in a safe location.



note

At this point, it is recommended for the wall to be finished, before proceeding. This will minimize interference of unit with trades, and keep remaining parts clean.

- 4 Attach frame and movement assembly to back box using hardware provided at locations shown. Move frame as necessary to reach all of the hardware. Tighten whiz flange screws using a 5/16" wrench.

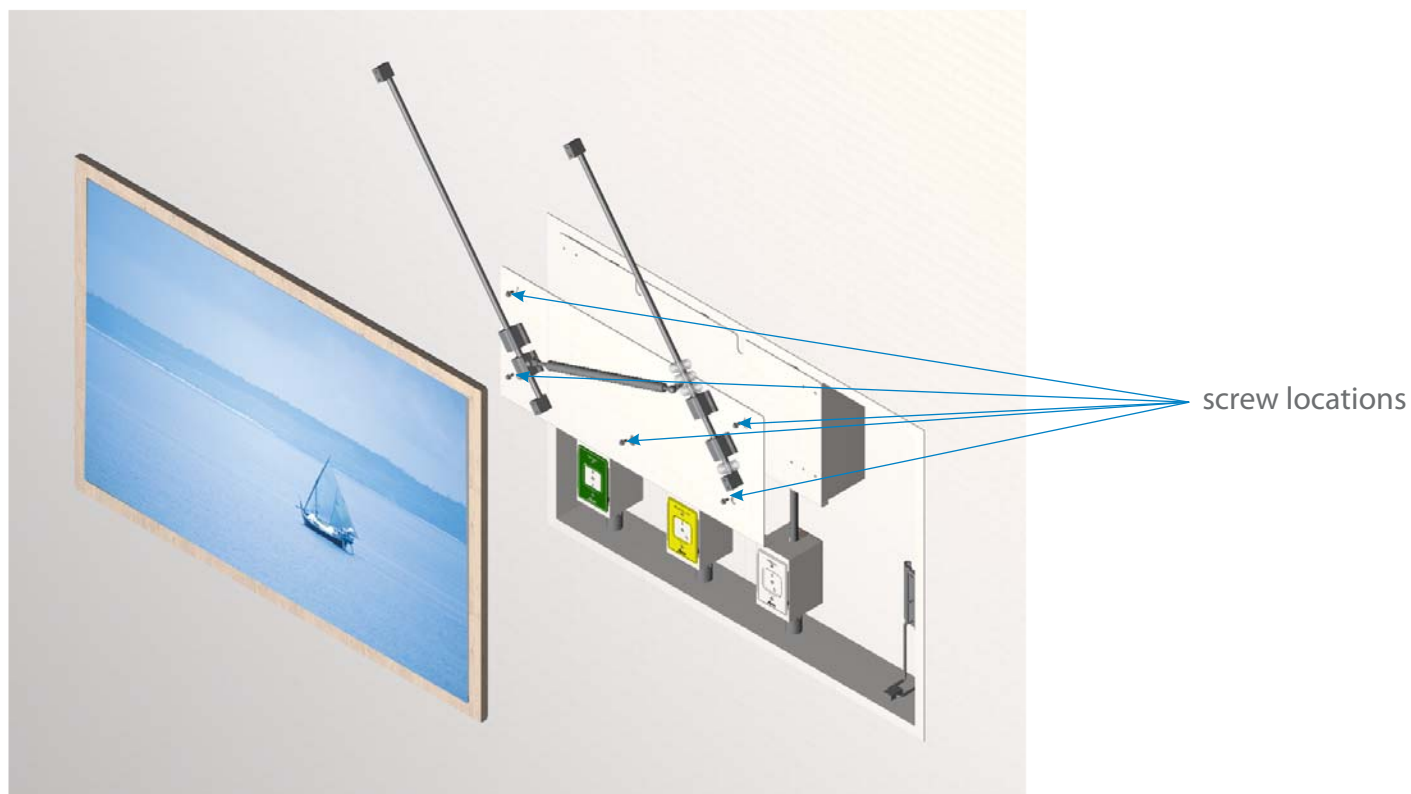

note

Frame shown removed to show parts behind. Do not remove!



85 Fulton Way, Richmond Hill
Ontario, L4B 2N4, Canada

31 Sprague Ave,
Amityville, NY 11701, USA

Toll Free Tel: 1.877.462.6426
Toll Free Fax: 1.866.440.4986
Tel: 905.764.0800 | Fax: 905.764.0862
www.amico.com